

### ID Panel Profile



1434



Blood and Transplant

**NHSBT REAGENTS Panel 2**

Product	Lot No.	Product	Lot No.	Product	Lot No.
ID Panel in Alsevers	R141 3561	ID Panel Papainised in Alsevers	R15 N/A	ID Panel in LISP	R14 N/A
ID Panel in CellStab	R142 3561	ID Panel Papainised in CellStab	R152 3561	Expiry Date : 2024.08.01	
ID Panel in CellMedia	R162 3561	ID Panel Papainised in CellMedia	R172 3561		

Patient's Name	Ref. No.	Sample No.	Conclusion
D.O.B.			Tested by Date

Unless otherwise indicated, all cells are positive for Kp<sup>b</sup> and Lu<sup>b</sup> and negative for Wr<sup>a</sup> and Co<sup>b</sup>.

Instructions for use can be found at <http://www.blood.co.uk/reagents>

EC REP Quality First International OÜ, Laki 30, 12915 Tallinn, Estonia

	Rh	C	D	E	c	e	C <sup>w</sup>	M	N	S	s	P1	Lu <sup>a</sup>	K	k	Kp <sup>a</sup>	Le <sup>a</sup>	Le <sup>b</sup>	Fy <sup>a</sup>	Fy <sup>b</sup>	Jk <sup>a</sup>	Jk <sup>b</sup>	Other													
1	R <sub>1</sub> <sup>w</sup> R <sub>1</sub>	+	+	0	0	+	+	+	0	+	0	3	0	+	0	0	0	+	0	+	0	+														
2	R <sub>1</sub> R <sub>1</sub>	+	+	0	0	+	0	0	+	0	+	0	0	0	+	0	+	0	+	0	+	0	Cob+													
3	R <sub>2</sub> R <sub>2</sub>	0	+	+	+	0	0	+	0	+	0	1	0	0	+	0	0	0	+	0	0	+														
4	r'r	+	0	0	+	+	0	0	+	0	+	2	0	0	+	0	0	+	0	+	0	+	Do(a-b+)													
5	r''r	0	0	+	+	+	0	0	+	0	+	4	+	0	+	0	0	+	0	+	+	0	HLA+													
6	rr	0	0	0	+	+	0	0	+	0	+	2	0	+	+	0	+	0	+	0	+	0	HLA+													
7	rr	0	0	0	+	+	0	+	+	0	+	0	0	0	+	0	0	+	0	0	+	0														
8	rr	0	0	0	+	+	0	0	+	+	0	0	0	0	+	+	+	0	0	+	0	+														
9	rr	0	0	0	+	+	0	+	0	+	0	0	0	0	+	0	0	0	+	0	+	0														
10	rr	0	0	0	+	+	0	+	0	0	+	0	+	0	+	0	0	+	+	0	0	+														
Reagent																																				
Lot No.																																				