

### ID Panel Profile



1434



Blood and Transplant

**NHSBT REAGENTS Panel 2**

Product	Lot No.	Product	Lot No.	Product	Lot No.
ID Panel in Alsevers	R141 3577	ID Panel Papainised in Alsevers	R15 n/a	ID Panel in LISP	R14 n/a
ID Panel in CellStab	R142 3577	ID Panel Papainised in CellStab	R152 3577	Expiry Date : 2025.03.13	
ID Panel in CellMedia	R162 3577	ID Panel Papainised in CellMedia	R172 3577		

Patient's Name	Ref. No.	Sample No.	Conclusion
D.O.B.			Tested by Date

Unless otherwise indicated, all cells are positive for Kp<sup>b</sup> and Lu<sup>b</sup> and negative for Wr<sup>a</sup> and Co<sup>b</sup>.  
Please note Cellstab cell 1 may be labelled as R142 4577.

Instructions for use can be found at <http://www.blood.co.uk/reagents>

EC REP Quality First International OÜ, Laki 30, 12915 Tallinn, Estonia

	Rh	C	D	E	c	e	C <sup>w</sup>	M	N	S	s	P1	Lu <sup>a</sup>	K	k	Kp <sup>a</sup>	Le <sup>a</sup>	Le <sup>b</sup>	Fy <sup>a</sup>	Fy <sup>b</sup>	Jk <sup>a</sup>	Jk <sup>b</sup>	Other													
1	R <sub>1</sub> <sup>w</sup> R <sub>1</sub>	+	+	0	0	+	+	+	0	+	0	3	0	+	0	0	0	+	0	+	0	+														
2	R <sub>1</sub> R <sub>1</sub>	+	+	0	0	+	0	0	+	0	+	0	0	0	+	0	+	0	+	0	+	0	Cob+													
3	R <sub>2</sub> R <sub>2</sub>	0	+	+	+	0	0	+	+	+	0	2	0	0	+	0	0	+	+	0	0	+														
4	r'r	+	0	0	+	+	0	0	+	0	+	1	0	0	+	0	+	0	0	+	0	+	Kna-													
5	r''r	0	0	+	+	+	0	0	+	0	+	2	0	0	+	0	+	0	0	+	+	0														
6	rr	0	0	0	+	+	0	0	+	0	+	4	0	+	+	0	+	0	+	0	+	0	Cob+HLA+													
7	rr	0	0	0	+	+	0	0	+	0	+	0	0	+	+	0	0	+	0	+	+	0														
8	rr	0	0	0	+	+	0	0	+	0	+	0	0	0	+	+	+	0	0	+	0	+														
9	rr	0	0	0	+	+	0	+	0	+	0	0	0	0	+	0	0	+	+	0	+	0														
10	rr	0	0	0	+	+	0	+	0	0	+	0	+	0	+	0	0	+	+	0	0	+														
Reagent																																				
Lot No.																																				