

## ID PANEL 2

|                       |           |                                  |           |                                                                                                |         |
|-----------------------|-----------|----------------------------------|-----------|------------------------------------------------------------------------------------------------|---------|
|                       | LOT       |                                  | LOT       |                                                                                                | LOT     |
| ID Panel in Alsevers  | R141 3591 | ID Panel Papainised in Alsevers  | R15 n/a   | ID Panel in LISP                                                                               | R14 n/a |
| ID Panel in CellStab  | R142 3591 | ID Panel Papainised in CellStab  | R152 3591 |  2025.09.25 |         |
| ID Panel in CellMedia | R162 3591 | ID Panel Papainised in CellMedia | R172 3591 |                                                                                                |         |

|        |                  |            |           |
|--------|------------------|------------|-----------|
| Name   | Hospital Number. | Conclusion | Tested by |
| D.O.B. | Sample No.       |            | Date      |

|    | Rh                                         | C | D | E | c | e | C <sup>w</sup> | M | N | S | s | P1 | Lu <sup>a</sup> | Lu <sup>b</sup> | K | k | Kp <sup>a</sup> | Kp <sup>b</sup> | Le <sup>a</sup> | Le <sup>b</sup> | Fy <sup>a</sup> | Fy <sup>b</sup> | Jk <sup>a</sup> | Jk <sup>b</sup> | Wr <sup>a</sup> | Co <sup>b</sup> | Other    |  |     |  |      |
|----|--------------------------------------------|---|---|---|---|---|----------------|---|---|---|---|----|-----------------|-----------------|---|---|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|----------|--|-----|--|------|
| 1  | R <sub>1</sub> <sup>w</sup> R <sub>1</sub> | + | + | 0 | 0 | + | +              | + | 0 | + | 0 | 3  | 0               | +               | 0 | + | 0               | +               | 0               | +               | +               | 0               | +               | 0               | 0               | 0               |          |  |     |  |      |
| 2  | R <sub>1</sub> R <sub>1</sub>              | + | + | 0 | 0 | + | 0              | + | + | 0 | + | 4  | 0               | +               | + | + | 0               | +               | +               | 0               | 0               | +               | 0               | +               | 0               | 0               |          |  |     |  |      |
| 3  | R <sub>2</sub> R <sub>2</sub>              | 0 | + | + | + | 0 | 0              | + | 0 | + | 0 | 0  | 0               | +               | 0 | + | 0               | +               | 0               | +               | 0               | +               | +               | 0               | 0               | 0               |          |  |     |  |      |
| 4  | r'r                                        | + | 0 | 0 | + | + | 0              | 0 | + | 0 | + | 0  | 0               | +               | 0 | + | 0               | +               | 0               | +               | 0               | +               | +               | +               | 0               | 0               |          |  |     |  |      |
| 5  | r''r                                       | 0 | 0 | + | + | + | 0              | 0 | + | 0 | + | 4  | 0               | +               | 0 | + | 0               | +               | 0               | 0               | +               | 0               | 0               | +               | 0               | 0               |          |  |     |  |      |
| 6  | rr                                         | 0 | 0 | 0 | + | + | 0              | + | 0 | + | 0 | 1  | 0               | +               | + | 0 | 0               | +               | 0               | +               | +               | 0               | +               | 0               | 0               | 0               | Do(a+b+) |  |     |  |      |
| 7  | rr                                         | 0 | 0 | 0 | + | + | 0              | 0 | + | 0 | + | 1  | 0               | +               | + | + | 0               | +               | +               | 0               | +               | 0               | 0               | +               | 0               | 0               |          |  |     |  |      |
| 8  | rr                                         | 0 | 0 | 0 | + | + | 0              | 0 | + | 0 | + | 4  | 0               | +               | 0 | + | +               | +               | 0               | +               | 0               | +               | +               | 0               | 0               | 0               | HLA+     |  |     |  |      |
| 9  | rr                                         | 0 | 0 | 0 | + | + | 0              | 0 | + | + | 0 | 4  | 0               | +               | 0 | + | 0               | +               | 0               | +               | 0               | +               | 0               | +               | 0               | 0               |          |  |     |  |      |
| 10 | rr                                         | 0 | 0 | 0 | + | + | 0              | + | 0 | 0 | + | 2  | +               | +               | 0 | + | 0               | +               | 0               | +               | 0               | w               | 0               | +               | 0               | 0               | HLA+     |  |     |  |      |
|    |                                            |   |   |   |   |   |                |   |   |   |   |    |                 |                 |   |   |                 |                 |                 |                 |                 |                 |                 |                 |                 |                 |          |  |     |  |      |
|    |                                            |   |   |   |   |   |                |   |   |   |   |    |                 |                 |   |   |                 |                 |                 |                 |                 |                 |                 |                 |                 |                 |          |  |     |  |      |
|    |                                            |   |   |   |   |   |                |   |   |   |   |    |                 |                 |   |   |                 |                 |                 |                 |                 |                 |                 |                 |                 |                 |          |  |     |  |      |
|    |                                            |   |   |   |   |   |                |   |   |   |   |    |                 |                 |   |   |                 |                 |                 | DAT             |                 | Anti-IgG        |                 | IgA             |                 | IgM             | C3c      |  | C3d |  | Ctrl |
|    |                                            |   |   |   |   |   |                |   |   |   |   |    |                 |                 |   |   |                 |                 |                 |                 |                 |                 |                 |                 |                 |                 |          |  |     |  |      |



blood.co.uk/reagents

NHSBT Reagents, 14 Estuary Banks,  
Speke, Liverpool, L24 8RB, Great BritainQuality First International OÜ,  
Laki 30, 12915 Tallinn, Estonia

0843



1434